

ZytoLight® SPEC ETV6 Dual Color Break Apart Probe



Background

The ZytoLight® SPEC ETV6 Dual Color Break Apart Probe (PL135) is intended to be used for the qualitative detection of translocations involving the human ETV6 gene at 12p13.2 in cytologic specimens by fluorescence *in situ* hybridization (FISH). The probe is intended to be used in combination with the ZytoLight® FISH-Cytology Implementation Kit (Prod. No. Z-2099-20).


The product is intended for professional use only. All tests using the product should be performed in a certified, licensed anatomic pathology laboratory under the supervision of a pathologist/human geneticist by qualified personnel.

The probe is intended to be used as an aid to the differential diagnosis of various cancers and therapeutic measures should not be initiated based on the test result alone.


Probe Description

The ZytoLight® SPEC ETV6 Dual Color Break Apart Probe is composed of:

- ZyGreen (excitation 503 nm/emission 528 nm) labeled polynucleotides (~10 ng/μl), which target sequences mapping in 12p13.2** (chr12:12,054,737-12,634,328) proximal to the ETV6 breakpoint region.
- ZyOrange (excitation 547 nm/emission 572 nm) labeled polynucleotides (~4.5 ng/μl), which target sequences mapping in 12p13.2** (chr12:11,393,774-11,808,608) distal to the ETV6 breakpoint region.
- Formamide based hybridization buffer

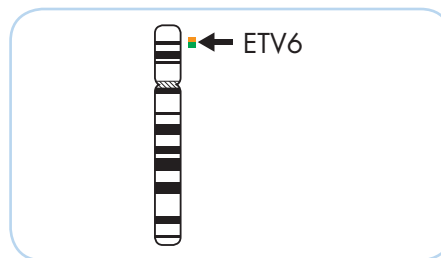


Ideogram of chromosome 12 indicating the hybridization locations.

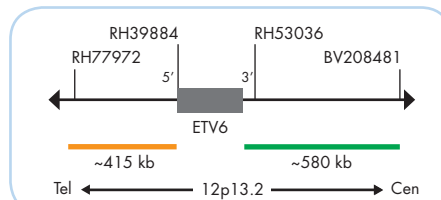


SPEC ETV6 Probe map (not to scale).

Results

In an interphase nucleus lacking a translocation involving the 12p13.2 band, two orange/green fusion signals are expected representing two normal (non-rearranged) 12p13.2 loci. A signal pattern consisting of one orange/green fusion signal, one orange signal, and a separate green signal indicates one normal 12p13.2 locus and one 12p13.2 locus affected by a translocation.

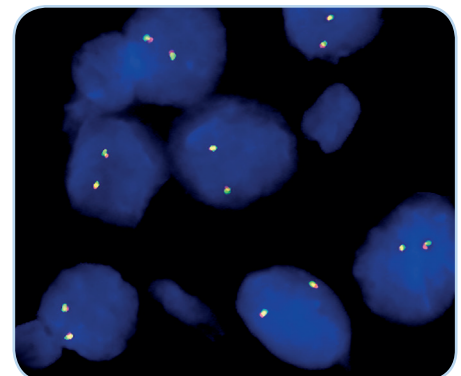

SPEC ETV6 Dual Color Break Apart Probe hybridized to normal interphase cells as indicated by two orange/green fusion signals per nucleus.

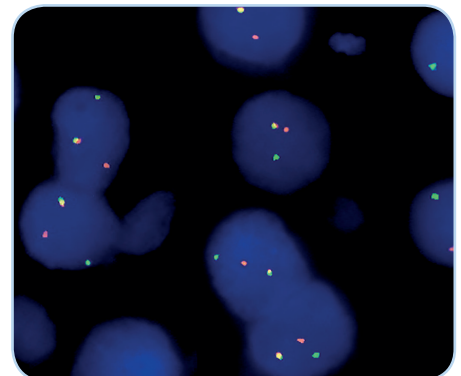

Example of an aberrant signal pattern: MASC tissue section of the salivary glands with translocation of the ETV6 gene as indicated by one non-rearranged orange/green fusion signal, one orange and one separate green signal.

Prod. No.	Product	Label	Tests* (Volume)
Z-2176-50	ZytoLight SPEC ETV6 Dual Color Break Apart Probe		5 (50 μl)
Z-2176-200	ZytoLight SPEC ETV6 Dual Color Break Apart Probe		20 (200 μl)
Related Products			
Z-2099-20	ZytoLight FISH-Cytology Implementation Kit Incl. Cytology Pepsin Solution, 4 ml; 20x Wash Buffer TBS, 50 ml; 10x MgCl ₂ , 50 ml; 10x PBS, 50 ml; Cytology Stringency Wash Buffer SSC, 500 ml; Cytology Wash Buffer SSC, 500 ml; DAPI/DuraTect-Solution, 0.8 ml		20

* Using 10 μl probe solution per test. labeled products are only available in certain countries. All other countries research use only! Please contact your local dealer for more information.

**According to Human Genome Assembly GRCh37/hg19