

ZytoLight® SPEC NTRK3 Dual Color Break Apart Probe



Background

The *ZytoLight*® SPEC NTRK3 Dual Color Break Apart Probe (PL164) is intended to be used for the qualitative detection of translocations involving the human NTRK3 gene at 15q25.3 in cytologic or formalin-fixed, paraffin-embedded specimens by fluorescence *in situ* hybridization (FISH). The probe is intended to be used in combination with *ZytoLight*® FISH Implementation Kits (Prod. No. Z-2028-5/-20, or Z-2099-20). The product is intended for professional use only. All tests using the product should be performed in a certified, licensed anatomic pathology laboratory under the supervision of a pathologist/human geneticist by qualified personnel. The probe is intended to be used as an aid to the differential diagnosis of various cancers and therapeutic measures should not be initiated based on the test result alone.


Probe Description

The *ZytoLight*® SPEC NTRK3 Dual Color Break Apart Probe is composed of:


- ZyGreen (excitation 503 nm/emission 528 nm) labeled polynucleotides (~10.0 ng/μl), which target sequences mapping in 15q25.3-q26.1** (chr15:88,825,346-89,475,889) distal to the NTRK3 breakpoint region.
- ZyOrange (excitation 547 nm/emission 572 nm) labeled polynucleotides (~4.5 ng/μl), which target sequences mapping in 15q25.3** (chr15:87,976,717-88,471,002) proximal to the NTRK3 breakpoint region.
- Formamide based hybridization buffer

Results


In an interphase nucleus of a normal cell lacking a translocation involving the 15q25.3-q26.1 band, two orange/green fusion signals are expected representing two normal (non-rearranged) 15q25.3-q26.1 loci. A signal pattern consisting of one orange/green fusion signal, one orange signal, and a separate green signal indicates one normal 15q25.3-q26.1 locus and one 15q25.3-q26.1 locus affected by a translocation. Isolated orange signals are the result of deletions distal to the NTRK3 breakpoint region or are due to unbalanced translocations affecting this chromosomal region.

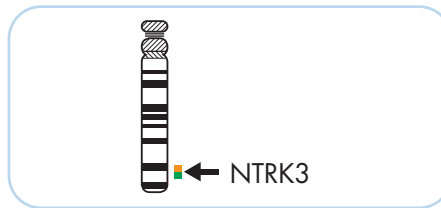


Ideogram of chromosome 15 indicating the hybridization locations.

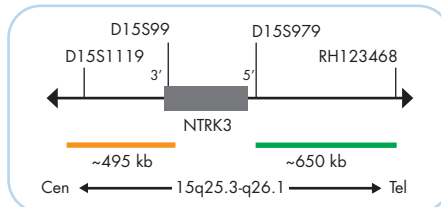

SPEC NTRK3 Probe map (not to scale).

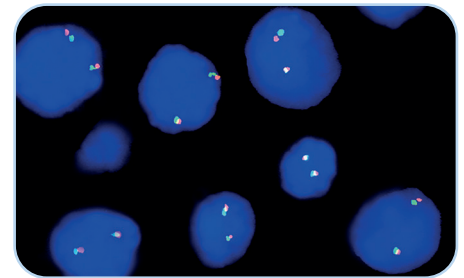

SPEC NTRK3 Dual Color Break Apart Probe hybridized to normal interphase cells as indicated by two orange/green fusion signals per nucleus.

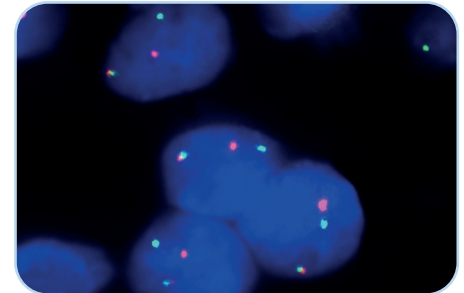

Example of an aberrant signal pattern: Secretory breast carcinoma tissue section with translocation affecting the 15q25.3-q26.1 locus as indicated by one non-rearranged orange/green fusion signal, one orange signal, and one separate green signal.

Prod. No.	Product	Label	Tests* (Volume)
Z-2206-50	<i>ZytoLight</i> SPEC NTRK3 Dual Color Break Apart Probe CE 0124 IVD	●/●	5 (50 μl)
Z-2206-200	<i>ZytoLight</i> SPEC NTRK3 Dual Color Break Apart Probe CE 0124 IVD	●/●	20 (200 μl)
Related Products			
Z-2028-5	<i>ZytoLight</i> FISH-Tissue Implementation Kit CE IVD Incl. Heat Pretreatment Solution Citric, 150 ml; Pepsin Solution, 1 ml; Wash Buffer SSC, 210 ml; 25x Wash Buffer A, 50 ml; DAPI/DuraTect-Solution, 0.2 ml		5
Z-2028-20	<i>ZytoLight</i> FISH-Tissue Implementation Kit CE IVD Incl. Heat Pretreatment Solution Citric, 500 ml; Pepsin Solution, 4 ml; Wash Buffer SSC, 560 ml; 25x Wash Buffer A, 100 ml; DAPI/DuraTect-Solution, 0.8 ml		20
Z-2099-20	<i>ZytoLight</i> FISH-Cytology Implementation Kit CE IVD Incl. Cytology Pepsin Solution, 4 ml; 20x Wash Buffer TBS, 50 ml; 10x MgCl ₂ , 50 ml; 10x PBS, 50 ml; Cytology Stringency Wash Buffer SSC, 500 ml; Cytology Wash Buffer SSC, 500 ml; DAPI/DuraTect-Solution, 0.8 ml		20

* Using 10 μl probe solution per test. IVD labeled products are only available in certain countries. All other countries research use only! Please contact your local dealer for more information.

**According to Human Genome Assembly GRCh37/hg19