

# Zyto *Dot*<u>CISH Implementation Kit</u>

**REF** C-3018-40



For use in chromogenic *in situ* hybridization (CISH) procedures

4250380N397Z



In vitro diagnostic medical device according to IVDR (EU) 2017/746

#### Intended use

The <u>Zyto Dot CISH Implementation Kit</u> is intended to be used in combination with Digoxigenin-labeled Zyto Dot Probes on formalin-fixed, paraffinembedded specimens by chromogenic *in situ* hybridization (CISH).

The product is intended for professional use only. All tests using the product should be performed in a certified, licensed anatomic pathology laboratory under the supervision of a pathologist/human geneticist by qualified personnel.

## 2. Test principle

The chromogenic *in situ* hybridization (CISH) technique allows the detection and visualization of specific nucleic acid sequences in cell preparations. Hapten-labeled nucleotide fragments, so called CISH probes, and their complementary target sequences in the preparations are co-denatured and subsequently allowed to anneal during hybridization. Afterwards, unspecific and unbound probe fragments are removed by stringency washing steps. Duplex formation of the labeled probe can be visualized using primary (unmarked) antibodies, which are detected by secondary polymerized enzyme-conjugated antibodies. The enzymatic reaction with chromogenic substrates leads to the formation of colored precipitates. After counterstaining the nucleus with a nuclear dye, hybridized probe fragments are visualized by light microscopy.

## 3. Reagents provided

The <u>ZytoDot CISH Implementation Kit</u> is available in one size and is composed of:

		Quantity	
Code	Component	₹ 40	Container
PT2	Heat Pretreatment Solution EDTA	500 ml	Screw-cap bottle (large)
ES1	Pepsin Solution	4 ml	Dropper bottle, white cap
WB1	Wash Buffer SSC	560 ml	Screw-cap bottle (large)
WB4	PBS/Tween	2x	Aluminium pack
BS1	Blocking Solution	4 ml	Dropper bottle, orange cap
AB1	Mouse_Anti-Dig	4 ml	Dropper bottle, pink cap
AB2	Anti-Mouse-HRP-Polymer	4 ml	Dropper bottle, violet cap
SB1a	DAB Solution A	0.3 ml	Dropper bottle, green cap
SB1b	DAB Solution B	10 ml	Dropper bottle, grey cap
CS1	Mayer's Hematoxylin Solution	20 ml	Screw-cap bottle, black
MT4	Mounting Solution (alcoholic)	4 ml	Glass bottle, brown
	Instructions for use	1	

<u>C-3018-40 (40 tests)</u>: Components ES1, BS1, AB1, AB2, SB1a-b, CS1, and MT4 are sufficient for 40 reactions. Component PT2 is sufficient for 7 staining jars of 70 ml each. Component WB1 is sufficient for 8 staining jars of 70 ml each. Component WB4 is sufficient for 28 staining jars of 70 ml each.

# 4. Materials required but not provided

- Zyto Dot CISH Probe
- Positive and negative control tissue
- Microscope slides, positively charged
- Water bath (80°C, 98°C)
- Hybridizer or hot plate
- Hybridizer or humidity chamber in hybridization oven
- Adjustable pipettes (10 μl, 1000 μl)
- Staining jars or baths
- Timer
- Calibrated thermometer
- Ethanol or reagent alcohol
- Xylene
- Methanol 100%
- Hydrogen peroxide (H<sub>2</sub>O<sub>2</sub>) 30%
- Deionized or distilled water
- Coverslips (22 mm x 22 mm, 24 mm x 32 mm)
- Rubber cement, e.g., <u>Fixogum Rubber Cement</u> (Prod. No. E-4005-50/-125) or similar
- Adequately maintained light microscope (400-630x)

# 5. Storage and handling

Store at  $2-8\,^{\circ}\text{C}$  in an upright position. Return to storage conditions immediately after use. Do not use reagents beyond expiry date indicated on the label. The product is stable until expiry date indicated on the label when handled accordingly.

### 6. Warnings and precautions

- Read the instructions for use prior to use!
- Do not use the reagents after the expiry date has been reached!
- This product contains substances (in low concentrations and volumes) that are harmful to health. Avoid any direct contact with the reagents.
   Take appropriate protective measures (use disposable gloves, protective glasses, and lab garments)!
- Report any serious incident that has occurred in relation to the product to the manufacturer and the competent authority according to local regulations!

P362+P364

- If reagents come into contact with skin, rinse skin immediately with copious amounts of water!
- A material safety data sheet is available on request for the professional user.
- Do not reuse reagents, unless reuse is explicitly permitted!
- Avoid cross-contamination of samples as this may lead to erroneous results.
- The specimens must not be allowed to dry during the hybridization and washing steps.

## Hazard and precautionary statements BS1, AB1, AB2, PT2, and WB1:

The hazard-determining component is a mixture of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H-isothiazol-3-one [EC no. 220-239-6] (3:1).

V	warning
H317	May cause an allergic skin reaction.
P261	Avoid breathing dust/fume/gas/mist/vapours/spray.
P272	Contaminated work clothing should not be allowed out of the workplace.
P280	Wear protective gloves/protective clothing/eye protection/face protection.
P302+P352	IF ON SKIN: Wash with plenty of water.
P333+P313	IF skin irritation or rash occurs: Get medical advice/attention.

## Hazards and precautionary statements for SB1a:

The hazard-determining component is biphenyl-3,3',4,4'tetrayltetraamine; diaminobenzidine.

Take off contaminated clothing and wash it before reuse.

N. S.	Dunger
H350	May cause cancer.
P201	Obtain special instructions before use.
P202	Do not handle until all safety precautions have been read and understood.
P280	Wear protective gloves/protective clothing/eye protection/face protection.
P308+P313	IF exposed or concerned: Get medical advice/attention.
P405	Store locked up.

# Hazards and precautionary statements for SB1b:

The hazard-determining component is imidazole; a reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H-isothiazol-3-one [EC no. 220-239-6] (3:1).



H317	May cause an allergic skin reaction.
H360D	May damage the unborn child.
P201	Obtain special instructions before use.
P261	Avoid breathing dust/fume/gas/mist/vapours/spray.
P280	Wear protective gloves/protective clothing/eye protection/face protection.
P302+P352	IF ON SKIN: Wash with plenty of water.
P308+P313	IF exposed or concerned: Get medical advice/attention.
P362+P364	Take off contaminated clothing and wash it before reuse.

#### Hazards and precaution statements for MT4:

The hazard-determining component is xylene.





#### Warning

H226	Flammable liquid and vapour.	
H312+H332	Harmful in contact with skin or if inhaled.	
H315	Causes skin irritation.	
H319	Causes serious eye irritation.	
H335	May cause respiratory irritation.	
H373	May cause damage to organs through prolonged or repeated exposure.	
P210	Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.	
P260	Do not breathe dust/fume/gas/mist/vapours/spray.	
P280	Wear protective gloves/protective clothing/eye protection/face protection.	
P305+P351+ P338	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.	
P337+P313	If eye irritation persists: Get medical advice/attention.	
P403+P235	Store in a well-ventilated place. Keep cool.	
EUH208	Contains methyl 2-methylprop-2-enoate; methyl 2-methylpropenoate; methyl methacrylate. May produce an allergic reaction.	

#### Hazards and precautionary statements for CS1 and WB4:

This product is not classified as hazardous according to Regulation (EC) No. 1272/2008.

## Special labeling of ES1:

EUH208 Contains Pepsin A. May produce an allergic reaction.
EUH210 Safety data sheet available on request.

# 7. Limitations

- For in vitro diagnostic use.
- For professional use only.
- For non-automated use only.
- The clinical interpretation of any positive staining, or its absence, must be done within the context of clinical history, morphology, other histopathological criteria as well as other diagnostic tests. It is the responsibility of a qualified pathologist/human geneticist to be familiar with the ISH probes, reagents, diagnostic panels, and methods used to produce the stained preparation. Staining must be performed in a certified, licensed laboratory under the supervision of a pathologist/human geneticist who is responsible for reviewing the stained slides and assuring the adequacy of positive and negative controls.
- Specimen staining, especially signal intensity and background staining, is dependent on the handling and processing of the specimen prior to staining. Improper fixation, freezing, thawing, washing, drying, heating, sectioning, or contamination with other specimens or fluids may produce artefacts or false results. Inconsistent results may result from variations in fixation and embedding methods, as well as from inherent irregularities within the specimen.
- The performance was validated using the procedures described in the instruction for use of the respective ZytoVision probe and implementation kit. Modifications to these procedures might alter the performance and have to be validated by the user. This IVD is only certified as CE when used as described in this instruction for use within the scope of the intended use.

## 8. Interfering substances

The following fixatives are incompatible with ISH:

- Bouin's fixative
- B5 fixative
- Acidic fixatives (e.g., picric acid)
- Zenker's fixative
- Alcohols (when used alone)
- Mercuric chloride
- Formaldehyde/zinc fixative
- Hollande's fixative
- Non-buffered formalin

# 9. Preparation of specimens

#### Recommendations:

- Avoid cross-contamination of samples in any step of preparation as this may lead to erroneous results.
- Fixation in 10% neutrally buffered formalin for 24 h at room temperature (18-25°C).
- Sample size  $\leq 0.5$  cm<sup>3</sup>.
- Use premium quality paraffin.
- Embedding should be carried out at temperatures lower than 65°C.
- Prepare 3-5 μm microtome sections.
- Use positively charged microscope slides.
- Fix tissue sections for 2-16 h at 50-60°C.

## 10. Preparatory treatment of the device

<u>PBS/Tween</u> (**WB4**) is to be pretreated according to the instructions in 11. "Assay procedure". All other kit reagents are ready-to-use. No reconstitution, mixing, or dilution is required.

#### 11. Assay procedure

#### 11.1 Day 1

#### Preparatory steps

- (1) Prepare an ethanol series (70%, 90%, and 100% ethanol solutions):
  Dilute 100% ethanol with deionized or distilled water. These solutions can be stored in suitable containers and can be re-used.
- (2) <u>Heat Pretreatment Solution EDTA</u> (PT2): Heat to 98°C in a covered staining jar.
- (3) Preparation of 3% H<sub>2</sub>O<sub>2</sub>: Dilute 1 part 30% H<sub>2</sub>O<sub>2</sub> in 9 parts 100% methanol.
- (4) ZytoDot CISH Probe: Bring to room temperature (RT) before use.

## Pretreatment (dewax/proteolysis)

- (1) Incubate slides for 10 min at 70°C (e.g., on a hot plate).
- (2) Incubate slides for 2x 5 min in xylene.
- (3) Incubate slides for 3x 3 min in 100% ethanol.
- (4) Incubate slides for 5 min in 3% H<sub>2</sub>O<sub>2</sub>.
- (5) Wash slides 2x 1 min in deionized or distilled water.
- (6) Incubate for 15 min in pre-warmed <u>Heat Pretreatment Solution EDTA</u> (PT2) at 98°C.

Use eight slides per staining jar (add dummy slides if needed).

- (7) Transfer slides immediately to deionized or distilled water and wash for 2x 2 min.
- (8) Apply (dropwise) <u>Pepsin Solution</u> (ES1) to the specimen and incubate for 5-15 min at 37°C in a humidity chamber.

**ES1** may form precipitates, which do not affect the quality. As a general rule, we recommend to ascertain the optimum time for proteolysis in pretests.

- (9) Immerse slides in deionized or distilled water.
- (10) Dehydration in: 70%, 90%, and 100% ethanol, each for 1 min.
- (11) Air dry sections.

Note: Make sure to completely dry sections prior to probe application.

# Denaturation and hybridization

- Pipette 10 µl of the Zyto Dot CISH Probe onto each pretreated specimen.
- (2) Cover specimens with a 22 mm x 22 mm coverslip (avoid trapped bubbles) and seal the coverslip.

We recommend using rubber cement (e.g., Fixogum) for sealing.

(3) Place slides on a hot plate or hybridizer and denature specimens for 5 min at 94-95°C. (4) Transfer slides to a humidity chamber and hybridize overnight at 37°C (e.g., in a hybridization oven).

It is essential that specimens do not dry out during the hybridization step.

# 11.2 Day 2

## Preparatory steps

- (1) Wash Buffer SSC (WB1): For stringency wash, heat to 80°C in a covered staining jar. WB1 may form precipitates at 2-8°C, which do not affect the quality and should dissolve when heated.
- (2) Preparation of Wash Buffer PBS/Tween: Add 1 tablet of <u>PBS/Tween</u> (WB4) to 1000 ml deionized or distilled water and dissolve.

Wash Buffer PBS/Tween is stable for one week when stored at 2-8°C.

(3) <u>Blocking Solution</u> (BS1), <u>Mouse-Anti-DIG</u> (AB1), <u>Anti-Mouse-HRP-Polymer</u> (AB2), <u>DAB Solution A</u> (SB1a), <u>DAB Solution B</u> (SB1b), <u>Mayer's Hematoxylin Solution</u> (CS1), <u>Mounting Solution</u> (alcoholic) (MT4): Bring to room temperature before use.

#### Post-hybridization and detection

- (1) Carefully remove the rubber cement or glue.
- (2) Remove the coverslip by submerging the slides in <u>Wash Buffer SSC</u> (WB1) at room temperature for 5 min.

WB1 can be reused once. Store at 2-8°C for a maximum of one week.

(3) Wash slides for 5 min in Wash Buffer SSC (WB1) at 80°C.

Use eight slides per staining jar (add dummy slides if needed).

- (4) Wash slides 2x 1 min in deionized or distilled water.
- (5) Immerse slides in Wash Buffer PBS/Tween.
- (6) Apply <u>Blocking Solution</u> (BS1) (1-2 drops per slide) to the slides and incubate for 10 min at RT.
- 7) Blot off Blocking Solution (BS1), but do not rinse!
- (8) Apply Mouse-Anti-DIG (AB1) (1-2 drops per slide) to the slides and incubate for 30 min at RT.
- (9) Wash slides 3x 1 min in PBS/Tween.
- (10) Apply Anti-Mouse-HRP-Polymer (AB2) (1-2 drops per slide) to the slides and incubate for 30 min at RT.
- (11) Wash slides 3x 1 min in PBS/Tween.
- (12) Prepare DAB Solution (working solution): fill 1 ml <u>DAB Solution B</u> (SB1b) in a graduated cup and add one drop (30 μl) <u>DAB Solution A</u> (SB1a). Mix well.
- (13) Apply DAB Solution (1-2 drops per slide) to the slides and incubate for 30 min at RT.
- (14) Transfer slides into a staining jar and wash 2 min under cold running tap water.
- (15) Counterstain specimens for 5-10 sec with <u>Mayer's Hematoxylin Solution</u> (CS1).
- (16) Transfer slides into a staining jar and wash 2 min under cold running tap water.
- (17) Dehydration in: 70%, 90%, and 100% ethanol, each for 1 min.
- (18) Incubate slides for 2x 2 min in xylene (use very pure xylene).
- (19) Avoiding trapped bubbles, cover the samples with a coverslip (22 mm x 22 mm; 24 mm x 32 mm) by using <u>Mounting Solution</u> (<u>alcoholic</u>) (MT4). Allow 20-30 min for the coverslip to become immobilized.

Using a pipette tip which has been cut off to increase the size of the opening, can make the pipetting process easier.

(20) Evaluate stained specimens by using light microscopy.

# 12. Interpretation of results

Using the ZytoDot CISH Implementation Kit, hybridization signals of Digoxigenin-labeled polynucleotides appear as brown- to dark brown-colored distinct dots. In interphases or metaphases of normal cells or cells without aberrations of the examined chromosomes, two signals per target will appear, except for probes targeting X or Y chromosomes, resulting in two, none or one signal depending on the gender and the probe used. In cells with chromosomal aberrations, a different signal pattern can be visible in interphases or metaphases. For more details on the interpretation of results, please refer to the instructions for use of the respective ZytoDot CISH Probe.

## 13. Recommended quality control procedures

Refer to the instructions for use of the respective ZytoVision probe.

# 14. Performance characteristics

Refer to the instructions for use of the respective ZytoVision probe. \\

# 15. Disposal

The disposal of reagents must be carried out in accordance with local regulations.

# 16. Troubleshooting

Any deviation from the operating instructions can lead to inferior staining results or to no staining at all. Please refer to <a href="www.zytovision.com">www.zytovision.com</a> for more information.

Weak signals or no signals at all

Possible cause	Action
Proteolytic pretreatment not carried out properly	Optimize pepsin incubation time, increase or decrease if necessary
Probe evaporation	When using a hybridizer, the use of the wet stripes/water filled tanks is mandatory. When using a hybridization oven, the use of a humidity chamber is required. In addition, the coverslip should be sealed completely, e.g., with Fixogum, to prevent drying-out of the sample during hybridization
Insufficient preparation of chromogenic substrate	Instead of using one drop of DAB Solution A use 30 $\mu$ l
Counterstaining time too long	Avoid dark counterstaining, because it may obscure positive staining signals
Bluing of counterstain not carried out properly	Use cold running tap water for bluing; do not use warm or hot water, or bluing reagents

Signals too strong

Possible cause	Action
Proteolytic pretreatment carried out too long	Optimize pepsin incubation time, increase or decrease if necessary
Substrate reaction is too intense	Shorten substrate incubation time; do not heat substrate solution over 25 °C; incubate at room temperature only

Signals fade or merge

Possible cause	Action
An unsuitable mounting solution has been used	Use only the mounting solution provided with the kit or xylene-based mounting solutions free of any impurities; do not use coverslip tape

Uneven or in some parts only very light staining

Possible cause	Action
Incomplete dewaxing	Use fresh solutions; check duration of dewaxing times
Reagent volume too small	Ensure that the reagent volume is large enough to cover the tissue area

# Inconsistent results

Possible cause	Action
Insufficient drying before probe application	Extend air-drying
Too much water/wash buffer on tissue prior to application of pepsin, antibodies and/or color substrates	Ensure that excess liquid is removed from tissue section by blotting or shaking it off the slide. Small amounts of residual water/wash buffer do not interfere with the test

Variations in tissue fixation and embedding methods	Optimize fixation and embedding methods
Variations in tissue section thickness	Optimize sectioning

Morphology degraded

Possible cause	Action
Cell or tissue sample has not been properly fixed	Optimize fixing time and fixative
Proteolytic pretreatment not carried out too long	Decrease pepsin incubation time

Cross hybridization signals; noisy background

Possible cause	Action
Sections dried out any time during or after hybridization	Avoid sections being dried out; use humidity chamber; seal coverslip properly
Prolonged substrate incubation time	Shorten substrate incubation time
Incomplete dewaxing	Use fresh solutions; check duration of dewaxing
Proteolytic pretreatment too strong	Optimize pepsin incubation time
Slides cooled to room temperature before hybridization	Transfer the slides quickly to hybridization temperature

Overlapping signals

Possible cause	Action
Inappropriate thickness of tissue sections	Prepare 3-5 $\mu$ m microtome sections

Specimen floats off the slide

Possible cause	Action
Proteolytic pretreatment too strong	Shorten pepsin incubation time

## 17. Literature

- Isola J, Tanner M (2004) *Methods Mol Med* 97: 133-44.
- Speel EJ, et al. (1994) J Histochem Cytochem 42: 1299-307.
- Tsukamoto T, et al. (1991) Int J Dev Biol 35: 25-32.
- Wilkinson DG: In Situ Hybridization, A Practical Approach, Oxford University Press (1992), ISBN 0 19 963327 4.

## 18. Revision

Revision	Description of the change
1.2.1	11. Assay procedure
	Wash Buffer PBS/Tween is stable for one week when stored at 2-8°C.
	www.zytovision.com

www.zytovision.com

Please refer to <a href="www.zytovision.com">www.zytovision.com</a> for the most recent instructions for use as well as for instructions for use in different languages.

Our experts are available to answer your questions. Please contact  $\underline{\mathsf{helptech@zytovision.com}}$ 



ZytoVision GmbH Fischkai 1 27572 Bremerhaven/ Germany Phone: +49 471 4832-300

Fax: +49 471 4832-509 www.zytovision.com Email: info@zytovision.com

# Trademarks:

ZytoVision® and Zyto*Dot*® are trademarks of ZytoVision GmbH.