

Zyto*Light* Glioma 1p/19q Probe Set

REF Z-2272-20 ∑ 20

For the qualitative detection of deletions involving the human chromosomal region 1p36.31 as well as deletions involving the human chromosomal region 19q13.32q13.33 by fluorescence *in situ* hybridization (FISH)



In vitro diagnostic medical device according to EU directive 98/79/EC

1. Intended use

The <u>ZytoLight Glioma 1p/19q Probe Set</u> is intended to be used for the qualitative detection of deletions involving the human chromosomal region 1p36.31 as well as deletions involving the human chromosomal region 19q13.32-q13.33 in formalin-fixed, paraffin-embedded specimens by fluorescence *in situ* hybridization (FISH). The probes are intended to be used in combination with the <u>ZytoLight FISH-Tissue</u> Implementation Kit (Prod. No. Z-2028-5/-20).

Interpretation of the results must be made within the context of the patient's clinical history with respect to further clinical and pathologic data of the patient by a qualified pathologist.

2. Clinical relevance

Deletions affecting the short arm of chromosome 1 (1p) and the long arm of chromosome 19 (19q) are frequently found in human gliomas. According to the 2016 WHO criteria for classification of tumors of the central nervous system, the detection of 1p/19q loss is required for the diagnosis of WHO grade II or III "oligodendroglioma, IDH-mutant and 1p/19q codeleted". Since both, astrocytomas and oligodendrogliomas, can exhibit IDH mutations, evaluation of 1p/19q status plays a critical role in differentiating astrocytoma from oligodendroglioma. Oligodendroglioma morphology, IDH-mutant genotype, and 1p/19q codeletion are associated with better response to chemotherapy and improved survival. Hence, determination of 1p and 19q status may aid in therapeutic decisions and predict outcome in patients with diffuse gliomas.

3. Test principle

The fluorescence *in situ* hybridization (FISH) technique allows for the detection and visualization of specific nucleic acid sequences in cell preparations. Fluorescently-labeled DNA fragments, so called FISH probes, and their complementary target DNA strands in the preparations are co-denatured and subsequently allowed to anneal during hybridization. Afterwards, unspecific and unbound probe fragments are removed by stringency washing steps.

After counterstaining the DNA with DAPI, hybridized probe fragments are visualized using a fluorescence microscope equipped with excitation and emission filters specific for the fluorochromes with which the FISH probe fragments have been directly labeled.

4. Reagents provided

The <u>ZytoLight Glioma 1p/19q Probe Set</u> is a set comprising two separate probes and a quenching solution:

- <u>ZytoLight SPEC 1p36/1q25 Dual Color Probe</u> (Prod. No. Z-2075-200)
- <u>Zyto*Light* SPEC 19q13/19p13 Dual Color Probe</u> (Prod. No. Z-2076-200)
- <u>ZyBlack Quenching Solution</u> (Prod. No. BS-0002-8)

The ZytoLight SPEC 1p36/1q25 Dual Color Probe (PL34) is composed of:

- ZyOrange (excitation 547 nm/emission 572 nm) labeled polynucleotides (~4.5 ng/µl), which target sequences mapping in 1p36.31* (chr1:5,808,946-6,176,336) (see Fig. 1).
- ZyGreen (excitation 503 nm/emission 528 nm) labeled polynucleotides (~10.0 ng/µl), which target sequences mapping in 1q25.3* (chr1:184,271,714-184,986,522) (see Fig. 1).
- Formamide based hybridization buffer

*according to Human Genome Assembly GRCh37/hg19

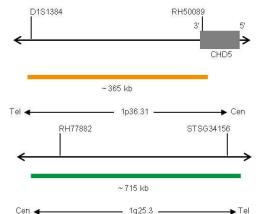


Fig. 1: Top: SPEC 1p36 Probe map; Bottom: SPEC 1q25 Probe map (not to scale)

The <u>ZytoLight SPEC 19q13/19p13 Dual Color Probe</u> (PL35) is composed of:

- ZyOrange (excitation 547 nm/emission 572 nm) labeled polynucleotides (~4.5 ng/µl), which target sequences mapping in 19q13.32-q13.33* (chr19:47,857,776-48,374,564) (see Fig. 2).
- ZyGreen (excitation 503 nm/emission 528 nm) labeled polynucleotides (~10.0 ng/µl), which target sequences mapping in 19p13.3* (chr19:658,555-1,144,465) (see Fig. 2).
- Formamide based hybridization buffer

*according to Human Genome Assembly GRCh37/hg19

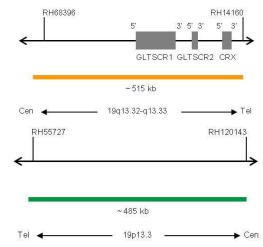


Fig. 2: Top: SPEC 19q13 Probe map; Bottom: SPEC 19p13 Probe map (not to scale)

Vers. 1.4 GB

The ZytoLight Glioma 1p/19q Probe Set is available in one size:

+ Z-2272-20: The individual probes are sufficient for 20 reactions of 10 μl each.

5. Materials required but not provided

- ZytoLight FISH-Tissue Implementation Kit (Prod. No. Z-2028-5/-20)
- <u>25x Wash Buffer A</u> (Prod. No.: WB-0002-50)
- Positive and negative control specimens
- Microscope slides, positively charged
- Water bath (37°C, 98°C)
- Hybridizer or hot plate
- Hybridizer or humidity chamber in hybridization oven
- Adjustable pipettes (10 μl, 25 μl)
- Staining jars or baths
- Timer
- Calibrated thermometer
- Ethanol or reagent alcohol
- XyleneDeionized or distilled water
- Deformation distinged water
 Coverslips (22 mm x 22 mm, 24 mm x 60 mm)
- Coversilps (22 mm x 22 mm, 24 mm x 00 mm)
 Rubber cement, e.g., <u>Fixogum Rubber Cement</u>
- (Prod. No. E-4005-50/-125) or similar
- Adequately maintained fluorescence microscope (400-1000x)
- Immersion oil approved for fluorescence microscopy
- Appropriate filter sets

6. Storage and handling

Store at 2-8°C in an upright position protected from light. Use protected from light. Return to storage conditions immediately after use. Do not use reagents beyond expiry date indicated on the label. The product is stable until expiry date indicated on the label when handled accordingly.

7. Warnings and precautions

- Read the instruction for use prior to use!
- Do not use the reagents after the expiry date has been reached!
- This product contains substances (in low concentrations and volumes) that are harmful to health and potentially infectious. Avoid any direct contact with the reagents. Take appropriate protective measures (use disposable gloves, protective glasses, and lab garments)!
- If reagents come into contact with skin, rinse skin immediately with copious quantities of water!
- A material safety data sheet is available on request for the professional user.
- Do not reuse reagents.
- Avoid cross-contamination of samples as this may lead to erroneous results.
- The probe should not be exposed to light, especially strong light, for a longer period of time, i.e., all steps should be accomplished, where possible, in the dark and/or using lightproof containers!

Hazard and precautionary statements:

Danaer

The hazard determining component is Formamide.



| • | |
|-------------------|--|
| H351 | Suspected of causing cancer. |
| H360FD | May damage fertility. May damage the unborn child. |
| H373 | May cause damage to organs through prolonged or repeated exposure. |
| P201 | Obtain special instructions before use. |
| P202 | Do not handle until all safety precautions have been read and understood. |
| P260 | Do not breathe dust/fume/gas/mist/vapours/spray. |
| P280 | Wear protective gloves/protective clothing/eye protection/face protection. |
| P308+P313 P405 | IF exposed or concerned: Get medical advice/attention. Store locked up. |
| | |

8. Limitations

- For *in vitro* diagnostic use.
- For professional use only.

• The clinical interpretation of any positive staining, or its absence, must be done within the context of clinical history, morphology, other histopathological criteria as well as other diagnostic tests. It is the responsibility of a qualified pathologist to be familiar with the FISH probes, reagents, diagnostic panels, and methods used to produce the stained preparation. Staining must be performed in a certified, licensed laboratory under the supervision of a pathologist who is responsible for reviewing the stained slides and assuring the adequacy of positive and negative controls.

• Specimen staining, especially signal intensity and background staining, is dependent on the handling and processing of the specimen prior to staining. Improper fixation, freezing, thawing, washing, drying, heating, sectioning, or contamination with other specimens or fluids may produce artefacts or false results. Inconsistent results may result from variations in fixation and embedding methods, as well as from inherent irregularities within the specimen.

• The probes should be used only for detecting loci described in 4. "Reagents provided".

• The performance was validated using the procedures described in this instruction for use. Modifications to these procedures might alter the performance and have to be validated by the user.

9. Interfering substances

Red blood cells present in the specimen might exhibit autofluorescence which hinders signal recognition.

The following fixatives are incompatible with FISH:

- Bouin's fixative
- B5 fixative
- Acidic fixatives (e.g., picric acid)
- Zenker's fixative
- Alcohols (when used alone)
- Mercuric chloride
- Formaldehyde/zinc fixative
- Hollande's fixative
- Non-buffered formalin

Treatment with organic solvents such as ethanol on day 1 after ZyBlack application and before hybridization will remove the ZyBlack staining effect and thus no reduction of autofluorescence will be visible.

10. Preparation of specimens

Recommendations:

- Fixation in 10% neutrally buffered formalin for 24 h at room temperature (18-25°C).
- Sample size ≤ 0.5 cm³.
- Use premium quality paraffin.
- Embedding should be carried out at temperatures lower than 65°C.
- Prepare 2-4 μ m microtome sections.
- Use positively charged microscope slides.
- Fix for 2-16 h at 50-60°C.

11. Preparatory treatment of the device

The product is ready-to-use. No reconstitution, mixing, or dilution is required. Bring probe to room temperature (18-25°C) before use, protect from light. Prior to opening the vial, mix by vortexing and spin down briefly.

12. Assay procedure

Specimen pretreatment

- 1. Bring <u>ZyBlack[™] Quenching Solution</u> to room temperature before use.
- Complete the specimen pretreatment (dewaxing, proteolysis) according to the instructions for use of the <u>ZytoLight FISH-Tissue</u> <u>Implementation Kit</u>.
- Apply an appropriate amount of <u>ZyBlack[™] Quenching Solution</u> on the air-dried specimen.
- 4. Incubate for 30 min at room temperature on a flat surface.

Vers. 1.4 GB

- 5. Wash 2x 5 min at room temperature in 1x Wash Buffer A (prepared as described in the instruction for use of the 25x Wash Buffer A).
- Wash 1x 1 min in deionized water. 6.
- Air-dry specimens for at least 30 min. 7.

Denaturation and hybridization

- Pipette 10 μ l of the probe onto each pretreated specimen. 1.
- Cover specimens with a 22 mm x 22 mm coverslip (avoid trapped 2. bubbles) and seal the coverslip.

We recommend using rubber cement (e.g., Fixogum) for sealing.

- 3. Place slides on a hot plate or hybridizer and denature specimens for 10 min at 75°C.
- Transfer slides to a humidity chamber and hybridize overnight at 4. 37°C (e.g., in a hybridization oven).

It is essential that specimens do not dry out during the hybridization step.

Post-hybridization

Perform post-hybridization processing (washing, counter-staining, fluorescence microscopy) according to the instructions for use of the Zyto Light FISH-Tissue Implementation Kit.

13. Interpretation of results


Zyto Light SPEC 1p36/1q25 Dual Color Probe:

With the use of appropriate filter sets, the hybridization signals of the probe appear orange (1p36 locus) and green (1q25 locus).

Normal situation: In interphases of normal cells or cells without a deletion involving the 1p36 locus, two green and two orange signals appear (see Fig. 3).

Aberrant situation: In a cell with deletion affecting the 1p36 locus, a reduced number of orange signals will be observed. Deletions affecting only parts of the 1p36 locus might result in a normal signal pattern with orange signals of reduced size (see Fig. 3).

Overlapping signals may appear as yellow signals.

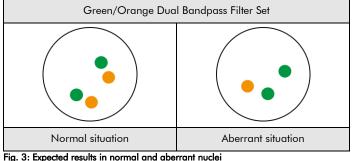

Zyto Light SPEC 19q13/19p13 Dual Color Probe:

With the use of appropriate filter sets, the hybridization signals of the probe appear orange (19q13 locus) and green (19p13 locus).

Normal situation: In interphases of normal cells or cells without a deletion involving the 19q13 locus, two green and two orange signals appear (see Fig. 4).

Aberrant situation: In a cell with deletions affecting the 19q13 locus, a reduced number of orange signals will be observed. Deletions affecting only parts of the 19q13 locus might result in a normal signal pattern with orange signals of reduced size (see Fig. 4).

Overlapping signals may appear as yellow signals.

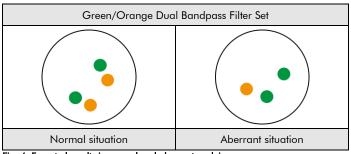


Fig. 4: Expected results in normal and aberrant nuclei

Other signal distribution may be observed in some abnormal samples which might result in a different signal pattern than described above, indicating variant rearrangements. Unexpected signal patterns should be further investigated.

Please note:

- Due to decondensed chromatin, single FISH signals can appear as small signal clusters. Thus, two or three signals of the same size, separated by a distance ≤ 1 signal diameter, should be counted as one signal.
- Do not evaluate overlapping nuclei.
- Do not count over-digested nuclei (recognized by dark areas visible inside of the nuclei).
- Do not count nuclei with strong auto-fluorescence, which hinders signal recognition.
- A negative or unspecific result can be caused by multiple factors (see chapter 17).
- In order to correctly interpret the results, the user must validate this product prior to use in diagnostic procedures according to national and/or international guidelines.

14. Recommended quality control procedures

In order to monitor correct performance of processed specimens and test reagents, each assay should be accompanied by internal and external controls. If internal and/or external controls fail to demonstrate appropriate staining, results with patient specimens must be considered invalid.

Internal control: Non-neoplastic cells within the specimen that exhibit normal signal pattern, e.g., fibroblasts.

External control: Validated positive and negative control specimens.

15. Performance characteristics

Accuracy: The location of hybridization of the probes was evaluated on metaphase spreads of a karyotypically normal male. In all tested specimens the probes hybridized solely to the expected loci. No additional signals or cross-hybridizations were observed. Therefore, the accuracy was calculated to be 100% for both probes.

Analytical sensitivity: For the analytical sensitivity assessment, the probes were evaluated on metaphase spreads of karyotypically normal males. All nuclei showed the expected normal signal pattern in all tested specimens. Therefore, the analytical sensitivity was calculated to be 100% for both probes.

Analytical specificity: For the analytical specificity assessment, the probes were evaluated on metaphase spreads of karyotypically normal males. In all tested specimens, all signals hybridized solely to the expected target loci and no other loci. Therefore, the analytical specificity was calculated to be 100% for both probes.

16. Disposal

The disposal of reagents must be carried out in accordance with local regulations.

17. Troubleshooting

Any deviation from the operating instructions can lead to inferior staining results or to no staining at all.

| Weak | signals | or | no | signals | at | all |
|------|---------|----|----|---------|----|-----|
| | | | | | | |

| theak signals of no signals at all | | |
|---|--|--|
| Possible cause | Action | |
| No target sequences available | Use appropriate controls | |
| Cell or tissue sample not fixed properly | Optimize fixing time and fixative or apply a post-fixation step as described in "assay procedure" of the manual of the <u>ZytoLight FISH-Tissue</u> Implementation Kit | |

| Heat pretreatment, proteolysis, denaturation, hybridization, or stringency wash temperature incorrect | Check temperature of all technical devices used, using a calibrated thermometer | | |
|--|---|--|--|
| Proteolytic pretreatment not carried out properly | Optimize pepsin incubation time, increase or decrease if necessary | | |
| Probe evaporation | When using a hybridizer, the use of the wet stripes/water filled tanks is mandatory. When using a hybridization oven, the use of a humidity chamber is required. In addition, the coverslip should be sealed completely, e.g., with Fixogum, to prevent drying-out of the sample during hybridization | | |
| Too low concentrated stringency wash buffer | Check concentration of stringency wash buffer | | |
| Old dehydration solutions | Prepare fresh dehydration solutions | | |
| Fluorescence microscope adjusted wrongly | Adjust correctly | | |
| Inappropriate filter sets used | Use filter sets appropriate for the fluochromes of the probe. Triple-bandpass filter sets provide less light compared to single or dual- bandpass filter sets. Consequently, the signals may appear fainter using these triple-bandpass filter sets | | |
| Photo-damage of the probes/fluorophores | Accomplish hybridization and washing steps in the dark | | |

Cross hybridization signals; noisy background

| Possible cause | Action | |
|--|--|--|
| Incomplete dewaxing | Use fresh solutions; check duration of dewaxing | |
| Proteolytic pretreatment too strong | Reduce pepsin incubation time | |
| Probe volume per area too high | Reduce probe volume per section/area, distribute probe dropwise to avoid local concentration | |
| Slides cooled to room temperature before hybridization | Transfer the slides quickly to 37°C | |
| Too high concentrated stringency wash buffer | Check concentration of stringency wash buffer | |
| Washing temperature following hybridization too low | Check temperature; increase if necessary | |
| Dehydration of specimens between the individual incubation steps | Prevent dehydration by sealing the slides and performing incubation in a humid environment | |

| Tissue morphology degraded | | |
|--|---|--|
| Possible cause | Action | |
| Cell or tissue sample has not been fixed properly | Optimize fixing time and fixative or apply a post-fixation step as described in "assay procedure" of the manual of the <u>Zyto<i>Light</i></u> <u>FISH-Tissue Implementation Kit</u> | |
| Proteolytic pretreatment not carried out properly | Optimize pepsin incubation time, increase or decrease if necessary | |
| Insufficient drying before probe application | Extend air-drying | |

Overlapping nuclei

| Possible cause | Action | |
|--|--|--|
| Inappropriate thickness of tissue sections | Prepare 2-4 μm microtome sections | |

Specimen floats off the slide

| Possible cause | Action |
|--|-------------------------------|
| Unsuitable slide coating | Use appropriate slides |
| Proteolytic pretreatment too strong | Reduce pepsin incubation time |

Weak counterstain

| Possible cause | Action |
|-----------------------------------|--|
| Low concentrated DAPI solution | Use <u>DAPI/DuraTect-Solution (ultra)</u> (Prod. No. MT-0008-0.8) instead |
| DAPI incubation time too short | Adjust DAPI incubation time |

18. Literature

- Barbashina V, et al. (2005) *Clin Cancer Res* 11: 1119-28.
- Cairncross JG, et al. (1998) J Natl Cancer Inst 90: 1473-9.
- Cairncross G, et al. (2013) *J Clin Oncol* 31: 337-43.
- Griffin CA, et al. (2006) J Neuropathol Exp Neurol 65: 988-94.
- Louis DN, et al. (ed.) (2016) WHO Classification of Tumours of the
- Central Nervous System (Revised 4th Edition).
- Reifenberger G, et al. (2017) Nat Rev Clin Oncol 14: 434-52.
- Rosenberg JE, et al. (1996) *Oncogene* 13: 2483-5.
- Smith JS, et al. (1999) *Oncogene* 18: 4144-52.
- Smith JS, et al. (2000) Genes Chromosomes Cancer 29: 16-25.
- Wilkinson DG: In Situ Hybridization, A Practical Approach, Oxford University Press (1992) ISBN 0 19 963327 4

Our experts are available to answer your questions. Please contact <u>helptech@zytovision.com</u>



ZytoVision GmbH Fischkai 1 27572 Bremerhaven/ Germany Phone: +49 471 4832-300 Fax: +49 471 4832-509 www.zytovision.com Email: info@zytovision.com

Trademarks: ZytoVision[®] and Zyto*Light[®]* are trademarks of ZytoVision GmbH.